

BacTerminator® Booster – the way to getting even more performance into your BacTerminator® Dental

BacTerminator® Booster is a specially designed unit which boost the performance of your BacTerminator® Dental. It secure a high continuous water flow even when multiple dental units are connected to a single BacTerminator® Dental.

BacTerminator® Booster is produced according to the Medical Device Directive 93/42/EEC and EN ISO 13485.



Fig. 1: BacTerminator® Booster

Data: BacTerminator® Booster systems:

	W x H x L mm	Weight kg	Connection water Inlet / Outlet, JG mm	Other external connection
BacTerminator® Booster	250 x 220 x 360	2,4	8 mm / 8 mm	None

Guidelines for installing the BacTerminator® Dental and the Booster:

	Instruments only	Instruments & mouth flush	Instruments, mouth flush & bowl flush	Numbers of chairs to flush at the same time (morning, start up)
BacTerminator® Dental EU	3 units	2 units	2 unit	1 unit
BacTerminator® Dental EU + Booster	8 units	6 units	3 units	3 units
BacTerminator® Dental DWGV	2 units	1 units	1 unit	1 unit
BacTerminator® Dental DVGW + Booster	6 units	3 units	2 units	2 units
BacTerminator® Dental Compact	2 units	1 units	1 unit	1 unit
BacTerminator® Dental Compact + Booster	6 units	3 units	2 units	2 units

Connection

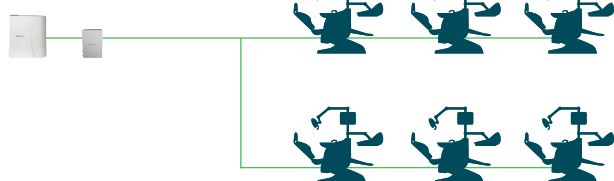


Fig. 2: BacTerminator® Dental Compact connected with BacTerminator® Booster

Never install a BacTerminator® Booster without consulting an authorized Adept Water Technologies dealer. Issues, such as types of connected units, distance from BacTerminator® Dental to units, risk of death legs etc. has to be taken into account to optimize the performance of the BacTerminator® Dental.

Adept Water Technologies always recommend that instrument and mouth flush water is connected to the BacTerminator® as Legionella from mouth flush system is known to have infected patients.